PATIENT GUIDE

Please consult the Aspira® System or Drainage System Patient Guide for warnings, cautions and full instructions for use.

For additional information, please visit:

www.myaspira.com
www.merit.com


STEP 1: Prepare the Workspace

- 1 Clear space on table.
- 2 Clean surface with disinfecting wipe.
- 3 Wash hands with warm soapy water. Scrub for one full minute. Dry with a clean towel.
- 4 Place supplies on cleaned area.

STEP 2: Drainage Procedure

NOTE: If the catheter is taped to the skin with tape strips, remove the tape.
NOTE: If the catheter is damaged, place the slide clamp (in kit) between the catheter damage and the exit site. Contact your doctor.

- 1 Peel open the pouch with the drainage kit.
- 2 Remove and discard valve cap from catheter valve.
- 3 Wipe catheter valve with alcohol pad.
- 4 Connect drainage line to catheter. You should hear or feel a click when secure.
- 5 Place bag on a flat surface at least arms length **below chest or abdomen**.
- 6 Squeeze pump one time. Let fluid drain until bag is full or fluid stops flowing. If fluid does not flow, refer to troubleshooting section.
- 7 When fluid stops or bag fills to 1,000 mL from the chest or from the abdomen, disconnect drainage line from catheter.
- 8 Wipe catheter valve with new alcohol pad.
- 9 Place new valve cap on catheter valve.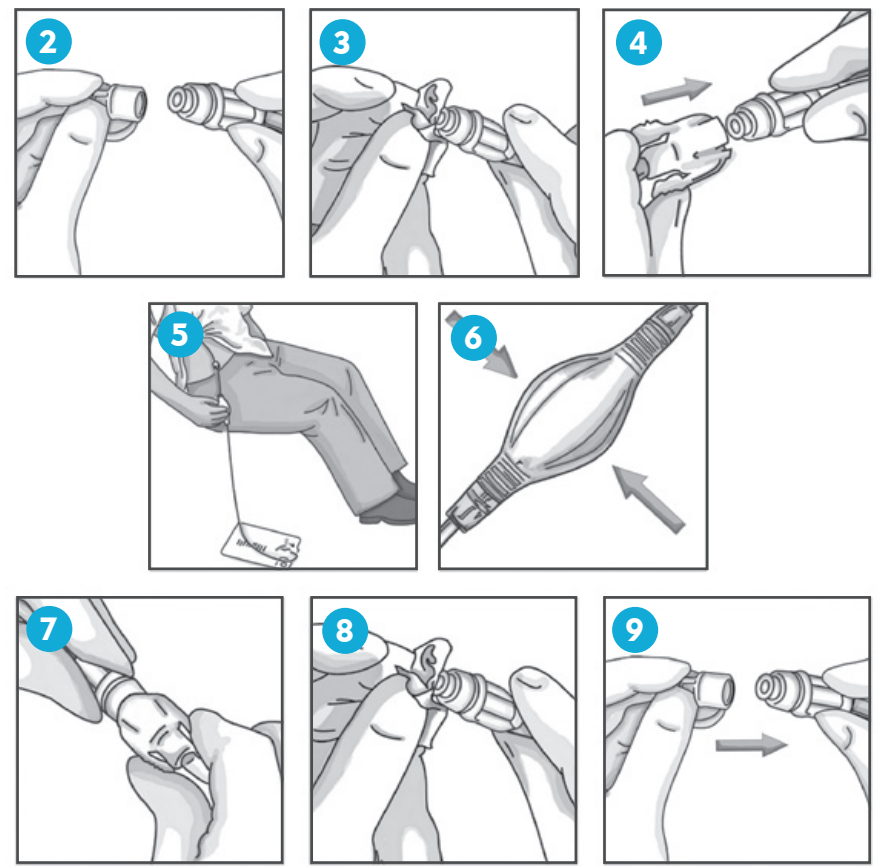

Remove dressing only if:


1. It has been one week since you changed the dressing.
- OR -
2. Your doctor recommended changing the dressing at every drainage.
- OR -
3. The area under the dressing is moist.

STEP 3: Dressing Procedure

If it is not time to change your dressing, tape the catheter to the skin and skip to step 4.

NOTE: Look at the exit site and skin around it. If you notice any redness, swelling, oozing or have pain at the exit site, finish draining and call your doctor.

- 1 Peel open the pouch with the dressing supplies.
- 2 Remove the gloves from the pouch.
- 3 Pick up glove at cuff end and place on hand.
- 4 Pick up second glove with the gloved hand and place it on your other hand.
- 5 Remove sterile sheet from the pouch and place it on prepared workspace.
- 6 Lay sheet flat so you can see dressing supplies.
- 7 Clean skin around exit site with alcohol pad.
NOTE: Make sure exit site and skin are dry before proceeding.
- 8 Place split gauze pad on skin around catheter.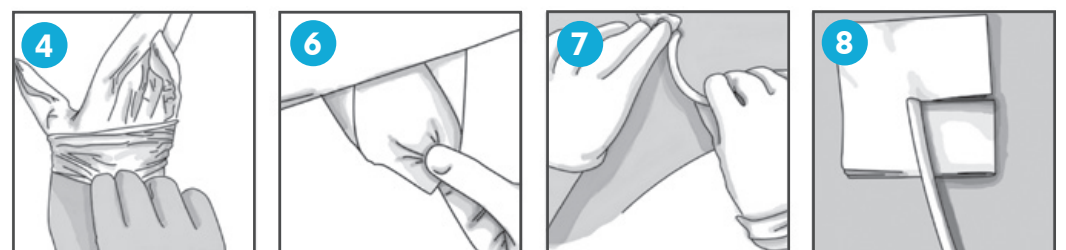


Dress At Every Drainage

- 1 Coil catheter on top of split gauze pad. Place gauze on top of the catheter.
- 2 Place clear dressing over catheter and gauze.

Dress Weekly

- 1 Place gauze on top of catheter. Place clear dressing over catheter and gauze.
- 2 Tape exposed catheter to skin.

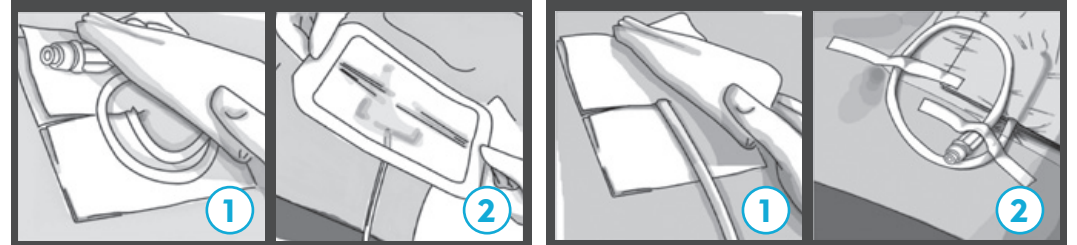
STEP 4: Discarding Fluid and Used Supplies

- 1 Over the toilet, tear or cut the corner of drainage bag.
- 2 Empty fluid into toilet.
- 3 Throw drainage bag and used supplies into trash.
- 4 Wash hands.

NOTE: If the catheter is damaged, place slide clamp provided in drainage kit between the damage and catheter exit site. Call your doctor.

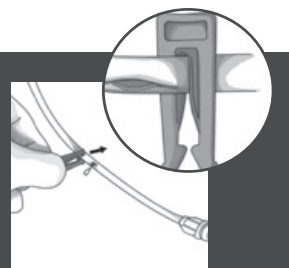CAUTION: If you have any of the following symptoms, complete the drainage and contact your doctor. Visit an emergency room or call 911.

- Redness, swelling, oozing or have pain around the exit site.
- Shortness of breath that isn't relieved after draining 1,000 mL from the chest or 2,000 mL from the abdomen at one time.
- On-going chest pain or shortness of breath but little or no fluid is draining from the catheter.
- Less than 25-50 mL drain during each drainage for 3 drainages in a row.
- The appearance of your fluid changes a lot between drainages.
- Body temperature above 100.5°F (38°C).

CAUTION: Do not drain more than 1,000 mL from the chest or 2,000 mL from the abdomen in any one drainage.

CAUTION: If fluid spills, clean using soap and warm water for clothing and skin and use bleach for household items.

NOTE: Do not recycle used drainage bags.

NOTE: If the catheter moves in and out of the exit site, call your doctor.

NOTE: If you feel dizziness, light headed or blurred vision, you may have low blood pressure. Stop draining. Call your doctor.

NOTE: Always follow the recommended cautions and orders of your healthcare provider.

**Drainage Kit
Reorder Number:**
4992301

**Dressing Kit
Reorder Number:**
4991503

Customer Service:
1-800-356-3748

Medical Support & Services:
1-833-3ASPIRA (327-7472)

TROUBLESHOOTING: If the flow of fluid does not start, check the connection. Gently squeeze the pump again. If this still does not work, try using a new drainage kit. If unable to start flow, contact your doctor.

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