Patient Information
on the
**ASPIRA PLEURAL
DRAINAGE SYSTEM**

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ASPIRA PLEURAL DRAINAGE CATHETER

The pain and difficult breathing you have felt is caused by extra fluid in your chest. Fluid is collecting between your lung and rib cage. The fluid pushes on your lungs making it hard for you to breathe. The fluid can also cause pain in your chest and make you cough. You now have a new treatment option that will allow you to take control of your symptoms.

The Aspira Pleural Drainage Catheter is a small tube that can be placed in your chest where the fluid is collecting. You will be able to drain this fluid at home to relieve the symptoms. You will not need to visit the doctor every time you feel the symptoms.

PK7631278
APPROXIMATE VOLUME
ml

Aspira[®]
DRAINAGE SYSTEM

1000
900
800
700
600
500
400
300
100
75

HERE
EMPTY

SOME COMMON QUESTIONS ABOUT THE ASPIRA PLEURAL DRAINAGE SYSTEM:

- **What will the Aspira Pleural Drainage catheter do for me?**

The catheter allows you to drain the extra fluid around your lungs at home. Draining fluid will help relieve symptoms such as chest pain and shortness of breath.

- **What will I have to do to take care of the catheter?**

You will drain fluid from the catheter when you feel pain or have difficulty breathing. Your doctor will tell you how often you should do this. You will want to keep the area around your catheter clean. Your physician will give you instructions on how this is done.

- **How will I know how to drain fluid?**

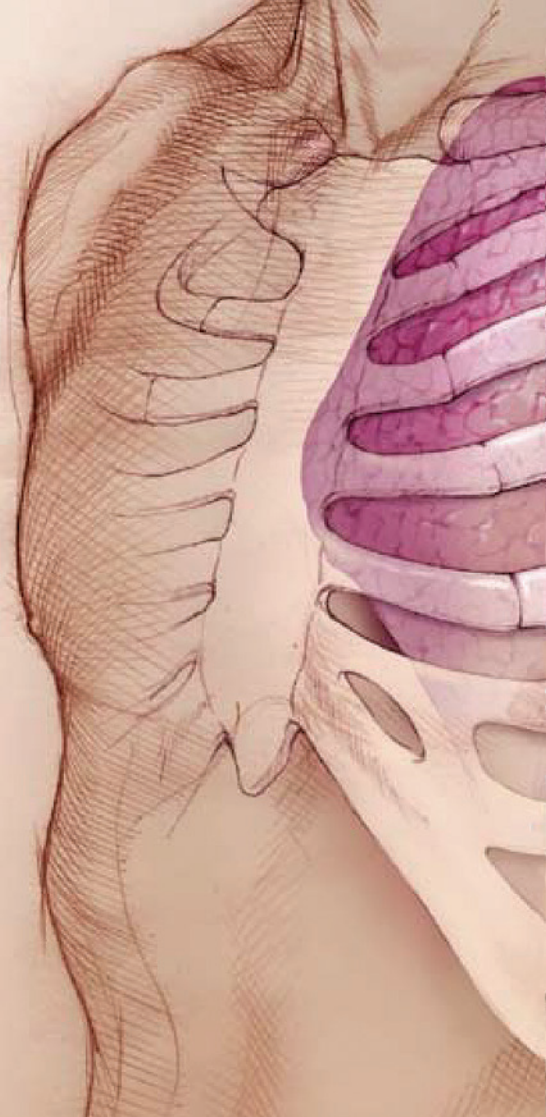
Your doctor or nurse will give you instructions on how to drain fluid. You should get a DVD that will teach you how to care for and drain fluid from your catheter.

- **Will I be able to shower with the catheter?**

You can shower as long as the clear dressing in the dressing kit is placed over the catheter, unless otherwise instructed by your doctor.

- **How long will the catheter be in my body?**

The catheter can stay in place for as long as it is needed. Your doctor will tell you more.

The Aspira Pleural Drainage Catheter is placed in your chest into a small space between your lungs and your ribs. This space is called the pleural space.

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Merit Medical Systems, Inc.